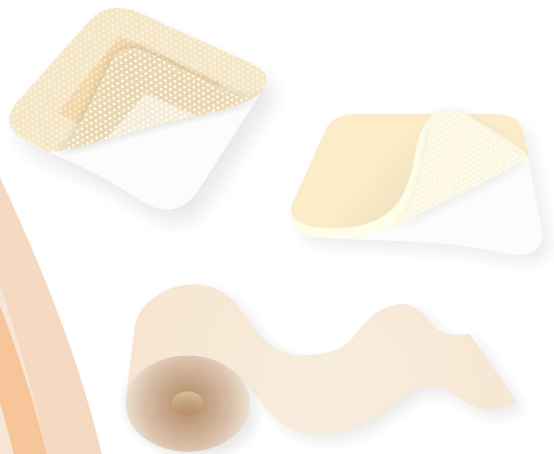
Silicone Dressing

Silicone Tape

Silicone Foam Dressing

Scar reducing emollient

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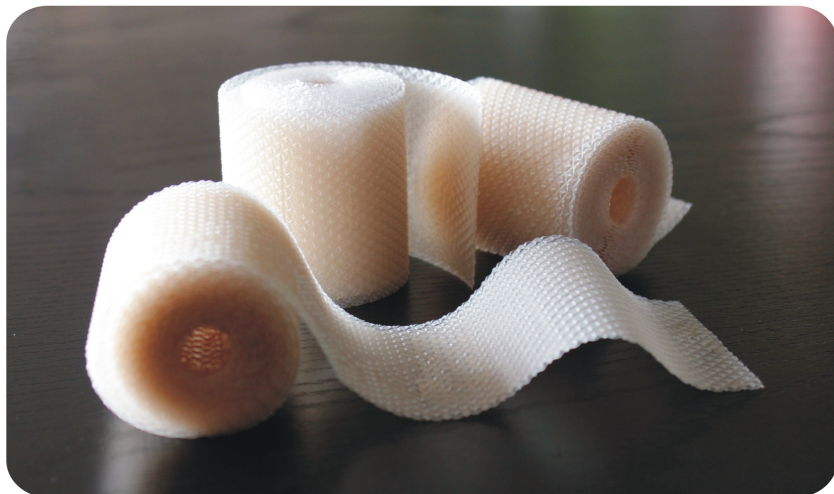
Silicone Tape

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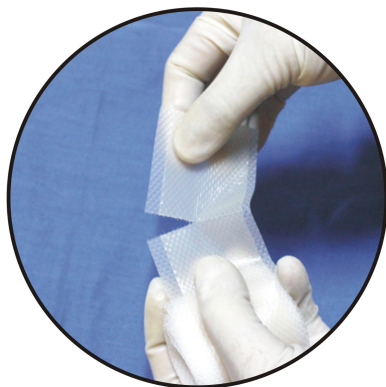
Product Features:

1. Gentle on the skin and painless to remove.
2. Reusable, does not stick to hair and leaves no residue, no remains.
3. Easy to use and hypoallergenic, with good adhesion.
4. Waterproof, but permeable.

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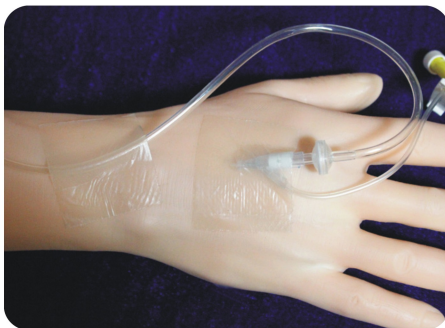
Intended Use:

1. Terrific for use on patients where skin condition is fragile, poor or damaged, such as patients after radiation therapy, chemotherapy, in dialysis or infants.
2. Applicable to a variety of in-dwelling catheters, electrodes, or other external fixation that requires gentle, but robust skin protection and adhesion.

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Silicone Foam Dressing

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Silicone
Foam Dressing

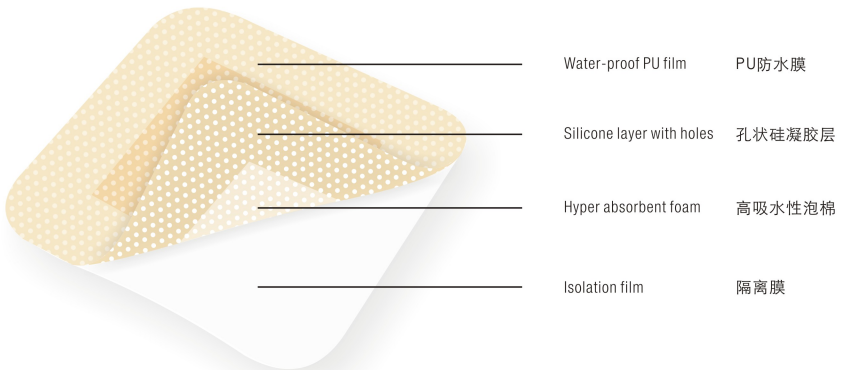
Structure illustration of Silicone Foam Dressing

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Structure illustration of Silicone Foam Dressing with border

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Product Features:

1. Gentle on the skin and painless to remove, with no residue.
2. "Vertically" absorbs wound mucosa exudate, reducing the risk of maceration around the wound.
3. Absorption capacity is great, this dressing keeps the wound under appropriate temperature and humidity according to basic "moist healing" principals.
4. Can be reused after inspection of the wound and will not adhere to the wound bed.
5. Does not damage the new epithelial cells when removed.
6. Helps to reduce scarring.
7. Waterproof, yet permeable. Showering is allowed while using this dressing.
8. Soft and comfortable, with good compression, works as a "cushion".

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Intended use:

1. Applicable to variety of acute and chronic wounds, especially highly exudating wounds.
2. Can be used in the prevention and treatment of various pressure ulcers.

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Cautions:

1. Coverage should be least 2 cm beyond the edge of the wound.
2. Not for use with oxidizing solutions, such as Hypochlorite solution or Hydrogen peroxide solution (H_2O_2).
3. Discontinue use if the patient is allergic to silicone.

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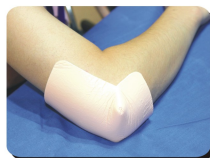
Silicone Foam Dressing (Heel and elbow)

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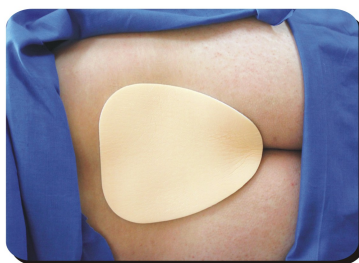
Silicone Foam Dressing (Shoulder Blade)

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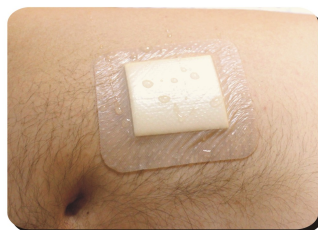
Silicone Foam Dressing with border

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Scar Reducing Emollient

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Product Features:

1. Effectively prevents the formation of scar
2. Softens the initial stage of scars and prevents further hyperplasia.
3. Washable and reusable.
4. Better effects against scarring with 3 months continuous use.
5. Can be cut-to-fit.

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Intended Use:

1. For all early stage wounds to help prevent scar formation.
2. Use on initial stages of wound tissue to soften scars and prevent further hyperplasia.

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1. Scar formation of early stage.
疤痕形成初期。

2. Paste the scar reducing emollient dressing on the wound.
将疤痕贴在创口处。

3. After three months ,the scar will heal without scar mark.
三个月后创口愈合且不留疤痕。

Scar Reducing Emollient 疤痕贴（减疤贴）

Caution:

1. Clean the skin and dressing once a day with warm water or neutral detergent.
2. Not for use with oxidizing solutions, such as Hypochlorite solution or Hydrogen peroxide solution (H_2O_2). Allow to air dry after cleaning.
3. Not for use on healed wounds.
4. First treatment should be between 2-4 hours, increasing by 4 hours daily, until 24 hour usage is attained, then use daily all day.
5. Discontinue use if the patient is allergic to silicone.

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Silicone Tape
Silicone Foam Dressing
Scar Reducing Emollient

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